


Mimitakara@myHearing App User Guide

Pairing


To pair your hearing device, turn the hearing device on by opening the battery door and then turn it off by closing the battery door. Follow the steps on screen.

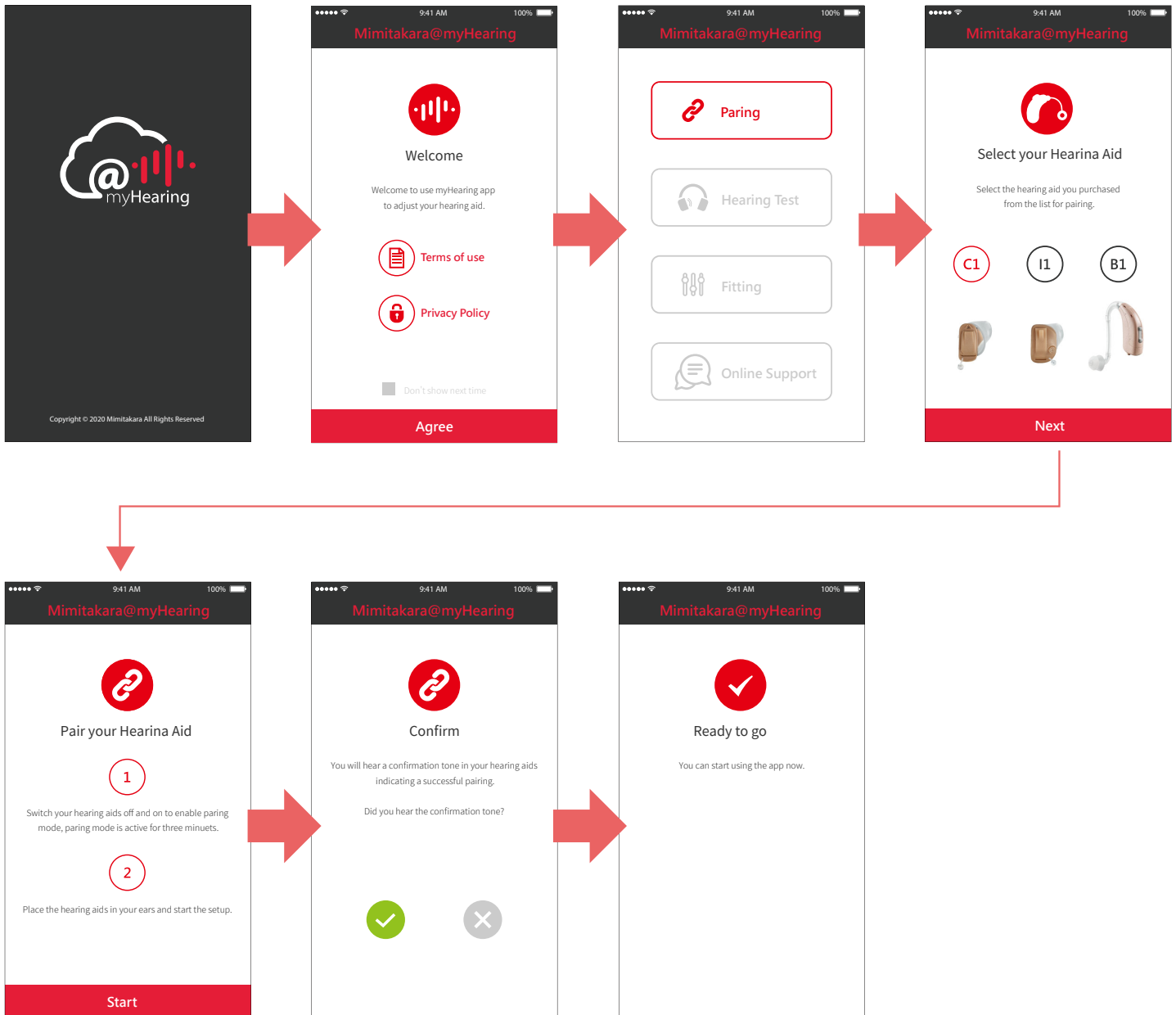

Hearing Test


To begin the test please complete user registration and login. This is only required for new user to log in for the first time. Follow the steps on screen.

If you are a new user, tap the “Register” to complete the process. Once you complete the registration, you will receive a verification code. Input your mobile phone number and verification code to log in and perform hearing test.

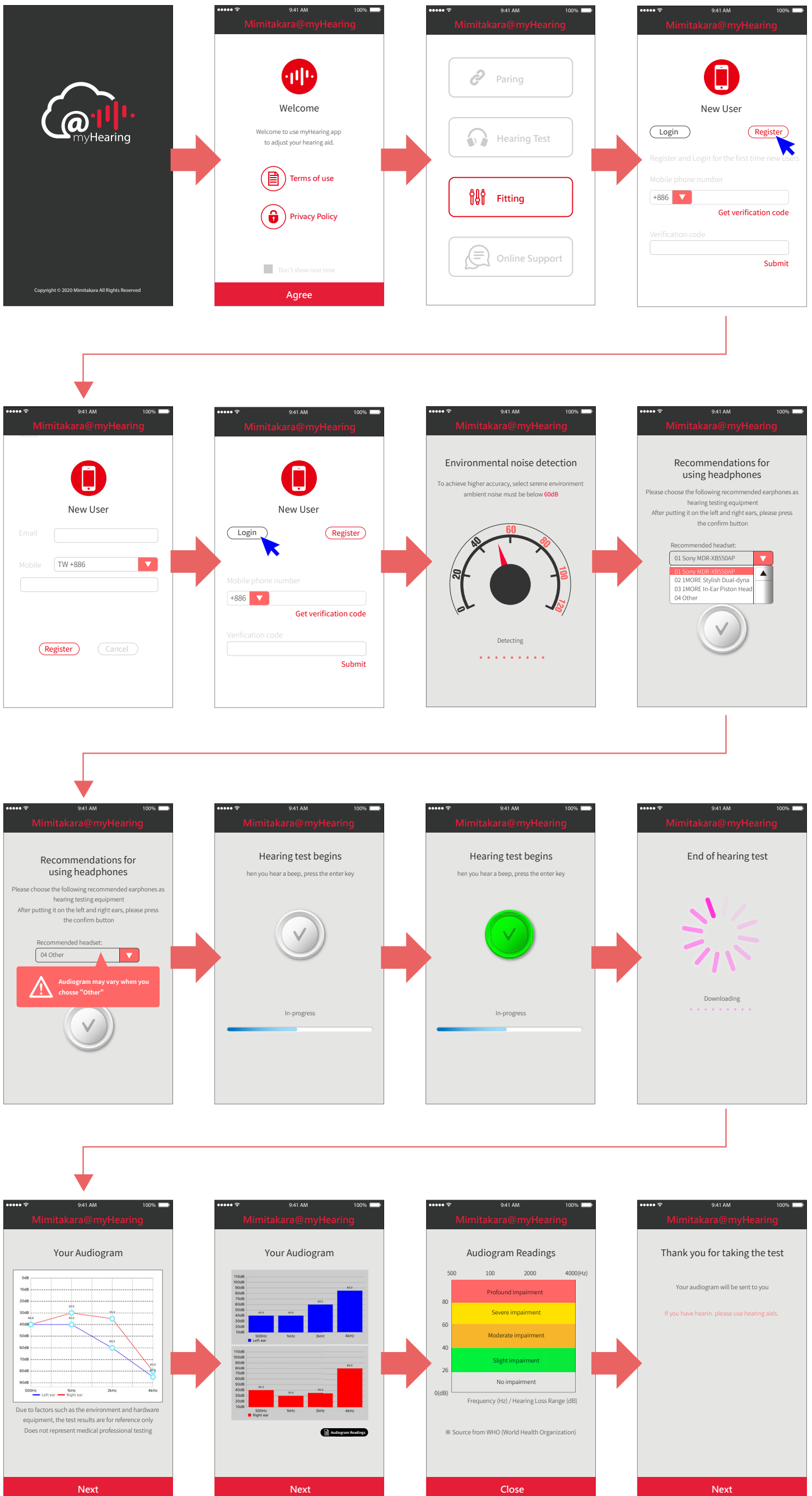

Fitting

Adjust volume levels and sound balance for right and left ear hearing devices.

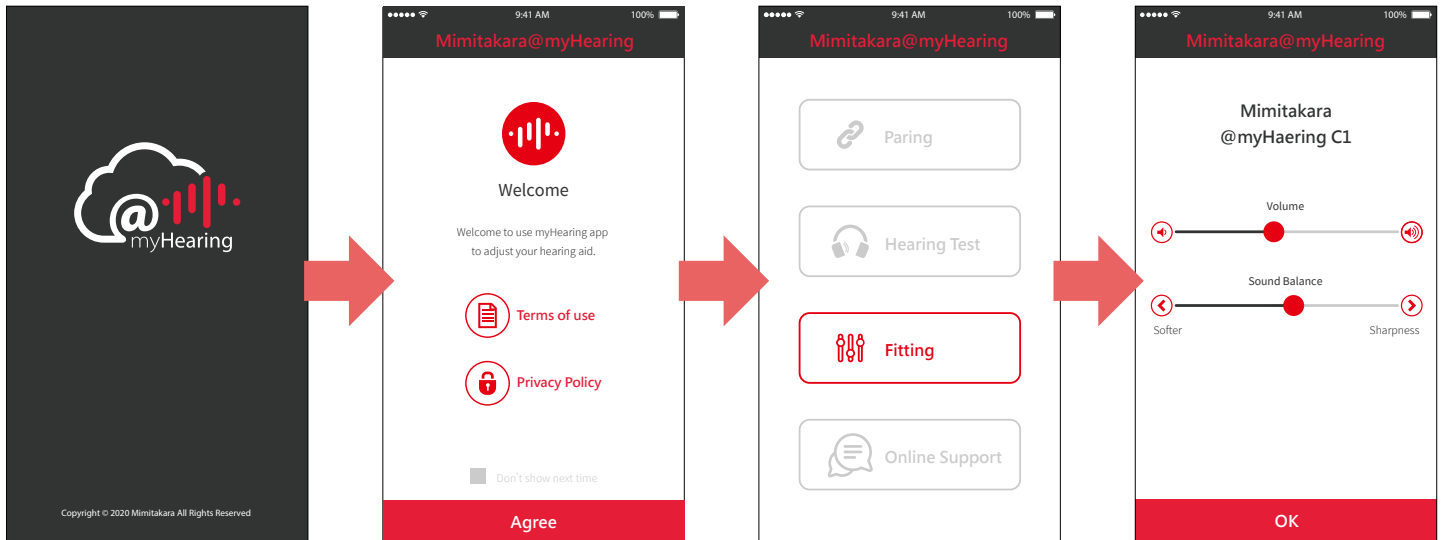

Online Support

Contact your Hearing Care Professional for further assistance by sending messages in the chat box or schedule a voice call service. After communication you will receive a SMS message containing a formula link, click the link your hearing instrument will be tuned automatically to best fit your hearing needs. If the adjustment doesn't fit your hearing need, go over the process again until it fits.

If you are a new user, tap the "Register" to complete the process. Once you complete the registration, you will receive a verification code. Input your mobile phone number and verification code to log in and get online support.

